



Tube Clamp Assembly

1211

Instructions For Use



1211A

1211S

1211T

1211T-W

1211MS-200

1211MS-400

1211A-OP

1211T-B

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings.
- It connects any pyramid system to a tube clamp
- The four set screws anchor the mating system and allow for angular alignment.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1211A	Aluminium	400mm	307g	158kg/350lbs
1211S	Stainless Steel	400mm	344g	136kg/300lbs
1211T	Titanium	400mm	306g	170kg/375lbs
1211T-W	Titanium	200mm	306g	170kg/375lbs
1211MS-200	Molded Stainless Steel	200mm	201g	90kg/198lbs
1211MS-400	Molded Stainless Steel	400mm	320g	90kg/198lbs
1211A-OP	Aluminium	400mm	280g	114kg/250lbs
1211T-B	Titanium	400mm	306g	170kg/375lbs


- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **M8 Alignment Set Screw:**
 - (Aluminum): **7-10Nm**,
 - (Titanium/Stainless Steel): **10-15Nm**
- **Thread Locker:** Apply Loctite once alignment and connection are finalized
- **CAUTION:** Do not clamp the tube in a vise or similar device, as this may damage the tube

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F).
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids.
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user.
- Do not reuse on another prosthesis.
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1211 IFU REV. A(3-26-2026)
Specifications subject to change.

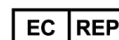

MDSS GmbH

Schiffgraben 41 | 30175 Hannover, Germany