

Tube Clamp, 30mm, Receiver 1212

Instructions For Use



1212A



1212T/T-W



1212S



1212MS



1212T-SH



1212T-B

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings.
- It connects any pyramid system to a pylon.
- The 1212T-SH is exclusively for the 1206T very short pylon to have a strong short segment with rotation and with pyramid receiver on both ends. Not intended for Aluminum pylons!.
- The four set screws anchor the mating system and allow for angular alignment.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1212A	Aluminium	18mm	83g	158kg/350lbs
1212T/T-W	Titanium	15mm	79g	170kg/375lbs
1212S	Stainless Steel	15mm	128g	136kg/300lbs
1212MS	Molded Stainless Steel	15mm	117g	90kg/198lbs
1212T-SH	Titanium	15mm	69g	170kg/375lbs
1212T-B	Titanium	15mm	79g	170kg/375lbs

- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **M8 Alignment Set Screw:**
 - (Aluminum): **7-10Nm**
 - (Titanium/Stainless Steel): **10-15Nm**
- **M5 Cap Screw:**
 - (Aluminum): **3-4Nm**
 - (Titanium/Stainless Steel): **5-7Nm**
- **Thread Locker:** Apply Loctite once alignment and connection are finalized

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F).
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids.
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user.
- Do not reuse on another prosthesis.
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1212 IFU REV. A (3-26-2026)

Specifications subject to change.