



Small Single Axis Ankle Adapter

Instructions For Use

1232



1232A



1232S

1. Application & Description

- This device is to be exclusively used for the prosthetic fitting of lower extremity amputations
- The articulating ankle joint features a plantar flexion stop made of an elastic member, providing an elastic plantar flexion stop.
- Only for sizes 21-25cm

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1232A	Aluminum	23mm	215g	90kg/198lbs
1232S	Stainless Steel	17mm	305g	100kg/220lbs

- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **Joint nut: 6 Nm**

3. Installation Overview

- **Mounting:** When mounting the ankle to the foot, tighten the provided nuts onto the threaded link of the ankle.
- **Torque Requirement:** Use a torque wrench to tighten nuts to exactly **6 Nm**.
- **Rotation Prevention:** You **must** insert the anti-rotation bridge between the two nuts to prevent them from rotating or loosening.
- **Note:** Tighten all screws that are positioned side-by-side using a torque wrench.

4. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F).
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids.
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user.
- Do not reuse on another prosthesis.
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1232 IFU REV.B (5-12-2026)

Specifications subject to change.