



Low Profile Tube Clamp

Instructions For Use

1241T



1241T

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings.
- It connects any pyramid system to a tube clamp.
- The four set screws anchor the mating system and allow for angular alignment.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1241T	Titanium	223mm	232g	170kg/375lbs

- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **M8 Alignment Set Screw:** 10-15Nm
- **Thread Locker:** Apply Loctite once alignment and connection are finalized.
- **CAUTION:** Do not clamp the tube in a vise or similar device, as this may damage the tube

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F)
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user
- Do not reuse on another prosthesis.
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1241T IFU REV. G (5-12-2026)

Specifications subject to change.