



Tube Clamp With Pyramid

Instructions For Use

1252



1252

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings
- The 1252 adapts a pyramid receiver to a tube.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1252S	Stainless Steel	8mm	105g	136kg/300lbs
1252T	Titanium	8mm	8g	170kg/375lbs

- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **M8 Alignment Set Screw (Stainless Steel/Titanium): 10-15 Nm**
- **M5 Cap Screw (Stainless Steel/Titanium): 5-7Nm**
- **Thread Locker:** Apply Loctite once alignment and connection are finalized

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F) 
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user
- Do not reuse on another prosthesis. 
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.

1252 IFU REV.B (5-12-2026)

Specifications subject to change.