



3-Prong, Stainless Steel Pyramid Connector

Instructions For Use

1255S



1255S

1. Application & Description

- The 1255S is to be used exclusively for the prosthetic fitting of lower extremity amputations.
- **Integration:** The device is laminate into the prosthetic socket for both transtibial and transfemoral
- **Compatibility:** The 3-Prong base is a fixed pyramid adapter.
- **Alignment:** The Pyramid should be set to the preferred alignment prior to lamination.
- **Finalization:** Apply Loctite to all screws once alignment and connections are finalized
- **Notice:** Always tighten screws using a calibrated torque wrench

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1255S	Stainless Steel	12mm	127g	100kg/220lbs

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F).
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments.
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids.
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments.
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user.
- Do not reuse on another prosthesis.
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



4. Reinforcement Recommendations

- The clinician is responsible for determining the adequate layup and reinforcement based on the specific patient's usage, activity level, and expected loading.

1255S IFU REV.H (5-12-2026)

Specifications subject to change.