

500 Foot – 1150

Rated for use with ST&G 500Lb. System

- Integrated ST&G 500Lb Rated Titanium SACH Adapter
- 10mm (3/8")
- 14mm (9/16") Integrated Foot Bolt
- Extra Firm Heel Cushion and Sturdy Toe Standard
- Rated up to 227kg / 500lb
- Sizes 26 - 30cm
- 86mm Build Height (27cm)
- 812g (27cm)

Ordering:

1150-size-side EX: 1150-27R (27cm Right)



The Heavy Duty SACH Foot is intended for single user for lower extremity only.

The Heavy Duty SACH Foot is designed for users who cannot use a standard weight rating SACH foot. This foot is not designed for high impact sports or activities, such as construction, or exposure to water.

Warning:

The Prosthetist is responsible for assessing the patient to be appropriate for this device. The Prosthetist must convey the relevant information for use and safety to the patient.

Patient should be instructed to return to their prosthetist immediately if they notice any unusual noise, motion, or change in performance that may occur from what could be a damaged module.

Warranty can be void due to:

Unusual usage or abuse (ex. excessive dusty environmental usage, exposure to water), Tampering with any fasteners, Modification to the module such as sanding or damaging the finish, patient weight too close to weight limit of foot module (we recommend a 50 lb. buffer).

Must be utilized with footwear, no bare foot usage!

500 pound SACH foot must be paired with 510T 500lbs BK Kit. Failure to do so could void warranty due to possible incompatibility of componentry.

Inspection:

Biannual interval of the Heavy Duty SACH foot is recommended: Check for signs of any wear, or damage of the wooden keel, or associated parts.

STORAGE

Operating & Storage Temperature Range:

-10°C to 50°C (14°F to 122°F)

Allowable relative humidity

0 % to 90 %, non-condensing

Store away from UV exposure



LIABILITY The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacture.

Environment:

Store at room temperature, and avoid extremely hot, cold, or moist storage environments Storage Temperature Range: between temperatures of -10°C to 50°C [14°F and 122°F]

Do not store in wet or very hot environment.

Allowable environmental conditions

Use – relative humidity: 0 % to 90 %, non-condensing

Storage/transportation – relative humidity: 20 % to 90 %, non-condensing

Splash - Fresh water, rain, salt water, urine, dust, sand, particles of foam cosmetics

cleaning required after contact with salt-laden air, salt water, chlorinated water, urine, sand

Unallowable environmental conditions

Storage/transportation: high vibrations, impacts

Cleaning agents containing solvents



CE Conformity

This product meets the requirements Council EU 2017/1745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines.

Please keep this manual in safe place for future use.



Useful Life

Service life of the product is covered by 2 year warranty period. Single user only, not to be reused on another user. Warranty and liability expires when device is transferred from original user/prosthesis, and liability will fall on the Prosthetist transferring device to another user/prosthesis.

Disposal

The device and its packaging must be disposed of in accordance with respective national/local environmental regulations.



Cleaning

Clean the product with a damp, soft cloth. Dry the product with a soft cloth. Allow to air dry in order to remove residual moisture. Replace if foot is submersed in water.

1150 IFU REV.B (07-12-24)