



# Double Pyramid Adapter

Instructions For Use

1271



1271

## 1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings.
- It connects any pyramid receiver to another receiver allowing for low build height and more angular adjustment when needed.

## 2. Technical Specifications

| P/N   | Material        | Build Height | Weight | Weight Limit |
|-------|-----------------|--------------|--------|--------------|
| 1271S | Stainless Steel | 10mm         | 139g   | 136kg/300lbs |
| 1271T | Titanium        | 10mm         | 77g    | 170kg/375lbs |

- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **M8 Alignment Set Screw:** 15 Nm
- **Thread Locker:** Apply Loctite once alignment and connection are finalized.

## 3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F) 
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user
- Do not reuse on another prosthesis. 
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.

1271 IFU REV.B (5-12-2026)

Specifications subject to change.