



4-Hole Threaded Base Plate

1223A & 1224

Instructions For Use



1223A



1224A



1224T

1. Application & Description

- The 1223A / 1224A / 1224T Base Plate is designed exclusively for prosthetic fittings of lower extremity amputations.
- It is suitable for both Transtibial and Transfemoral sockets.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1223A	Aluminium	8mm	65g	125kg/275lbs
1224A	Aluminium	8mm	35g	125kg/275lbs
1224T	Titanium	8mm	66g	170kg/375lbs

- **Recommended Torque:**
- **M6 4-Hole Mounting Screw Torque:**
 - (1223): 10Nm
 - (1224): 10Nm

3. Installation Overview

- **Attachment:** The plate is anchored to the prosthetic socket using a 4-hole connector system and four socket mounting screws.
- **Compatibility:** This device can be used with the following ST&G components:
 - **4 Hole Rotating Base:** 1214T and 1214S
 - **4 Hole Pattern Adapters:** 1217T/S and 1218T/S
- **Fastening:** Tighten all screws that are side by side using a torque wrench

4. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F)
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids.
- **Storage:** Store at room temperature; avoid wet; very hot or extremely cold environments.
- **Warranty:** 2 years, provided it is used as intended without modification and only the original user.
- Do not reuse on another prosthesis
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1223A_1224 IFU REV. B (5-12-2026)
Specifications subject to change.