



# Diagonal Offset 4-Hole Pyramid 1727

## Instructions For Use



1727

### 1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings.
- It connects any 4-Hole to a pyramid receiver like a pylon pyramid receiver.
- The 1727 4-Hole Pyramid has a combination 5 mm lateral, and 7 mm diagonal offset to help correct misalignment.

### 2. Technical Specifications

| P/N   | Material        | Build Height | Weight | Weight Limit |
|-------|-----------------|--------------|--------|--------------|
| 1727S | Stainless Steel | 9mm          | 125g   | 136kg/300lbs |
| 1727T | Titanium        | 9mm          | 92g    | 170kg/375lbs |

- **Recommended Torque:**
- **M6 Mounting Screw:** 7-10Nm
- **Thread Locker:** Apply Loctite once alignment and connection are finalized.

### 3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F).
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids.
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments.
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user.
- Do not reuse on another prosthesis.
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1727 IFU REV. B (5-12-2026)

Specifications subject to change.